ARTNR 111521

Infusionsautomat

Infusion Thermo Bag | Infusionsautomat

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CONTENTS

General information

- 21 Purpose
 - 21 Safety – regulations and warnings
 - 22 Markings and symbols
 - 23 Certificates and markings
 - 23 Technical lifespan
 - 24 Recycling
 - 24 Manufacturer
-

Instruction manual

- 25 Unpacking
 - 25 Usage
 - 26 Cleaning and maintenance
 - 26 Safety function
 - 27 Trouble shooting
-

Product description

- 29 Functional description
 - 31 Technical data
-

Accessories and spare parts

- 32 Content Installation kit (article number 009122)
- 32 Mounting instructions for Installation kit
- 33 Operational control

General information

PURPOSE

This manual is intended to provide necessary information to users to ensure proper and safe use of the product.

SAFETY – REGULATIONS AND WARNINGS

Read the entire user manual and follow the instructions before using the product for the first time. The user manual has been written to ensure that you can handle the product in a safe and correct manner.

- » The product may only be used by authorised personnel with relevant competence to operate the product.
- » The product is not intended for outdoor use and therefore not designed for use in such an environment.
- » The product must be installed by qualified personnel according to the instructions in the user manual.
- » Never use the product near a mobile phone, CB radio or any other form of radio communications and/or electromagnetic field as this can affect the product's function.
- » The product is not intended for storage of blood/plasma or heat-sensitive infusion solutions.
- » When removing the infusion solution from the bag, the user must keep the heated bag against the skin as extra safety

check to ensure body temperature as a final test that the temperature of the infusion solution is correct.

- » If the display gives off a red light, do not remove the infusion solution bag for use.
- » If the display does not illuminate, the bag should be taken out of service and troubleshooting be initiated.
- » The user is responsible for following the regulations for storage, heating, durability and handling provided by the manufacturer of the infusion solutions.

MARKINGS AND SYMBOLS

Explanation of markings and symbols on the product:

Manufacturer

Direct current

Car battery

SN

Serial number

Manufacture date

The product is CE marked according to the medical device directive (MDD) 93/42/EEC in class IIa

Warning, read attached documentation

Not intended for outdoor use

The product must not be disposed of in the refuse but must be recycled. Used bag is to be deposited in accordance with WEEE Directive 2002/96/EC for disassembly and recycling.

CERTIFICATES AND MARKINGS

The product is tested by Intertek in accordance with IEC 60601-1. CE marked according to the medical device directive (MDD) Annex V 93/42/EEC in class IIa.

TECHNICAL LIFESPAN

This product has a technical lifespan, which is determined

by Sacci Ryggsäckar AB to be 2 years. Like all products, the infusion thermo bag from Sacci Ryggsäckar AB is subjected to wear and ageing, and even if the product is maintained and service is performed regularly, the function cannot be guaranteed after the specified lifespan. Service, spare parts, et cetera that are provided by the manufacturer after the expiry of the specified lifespan does not extend the lifespan.

RECYCLING

Used bag is to be deposited in accordance with WEEE Directive 2002/96/EC for disassembly and recycling.

THIS PRODUCT IS MANUFACTURED, MARKETED AND DISTRIBUTED BY:

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Instruction manual

UNPACKING

1. Unpack and inspect the product upon delivery. Contact your local distributor immediately if you find any damage to the product.
2. If you need to change the adapter, follow the instructions under the heading Electrical connection.
3. Connect the product to a power source.

USAGE

1. Place three 1 L infusion solution bags on top of each other (stacked) and then close the bag.
2. Wait until the display lights green before the infusion solution bag can be removed to ensure that the temperature is $37\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$. Filled infusion thermo bag has a heating time of about 2 hours.
3. When the display lights green, the infusion solution bag can be removed from the lower opening of the infusion thermo bag by opening the zipper.
4. Check the removed infusion solution bag temperature by holding a hand to it to ensure that the content is at body temperature before administering the infusion solution to the patient.

5. New unheated infusion solution bags are placed inside the product from above. Follow local routines for labelling, storage, handling and disposal of infusion solution bags.

CLEANING AND MAINTENANCE

In case of blood or vomiting on/in the product, first rinse with water and then scrub/wipe with disinfectant.

In order to wash the fabric part of the product, first the metal insert with associated electronics needs to be removed by technical personnel. The outer bag in fabric may be washed in a washing machine at 40 °C, but they must not be centrifuged or tumble-dried. Instead, let the bag drip dry.

SAFETY FUNCTION

The product has a built-in overheat protection that shuts off the product in case the temperature control stops working properly. In the event of a triggered overheating protection, the product must be taken out of operation and undergo an operational control. For details, read under the heading Operational control. In case the operational control does not give the desired outcome, the product must be sent to Sacci Ryggsäckar AB for repair.

TROUBLE SHOOTING

Symptom	Possible causes	Action
<i>Display does not light up.</i>	Overheat protection triggered.	Take the product out of service and perform an operational control. In the event of non-approved operational control, the product must be sent to Sacci Ryggsäckar AB for repair.
	Cord between control board and display disconnected.	Check to make sure the cord is properly attached.
<i>Red or green display without colour change despite changed temperature.</i>	Faulty display.	Take the product out of service and send to Sacci Ryggsäckar AB for repair.

Symptom	Possible causes	Action
<i>Display lights up but no heat is supplied.</i>	Faulty control unit or heating circuit.	Take the product out of service and send to Sacci Ryggsäckar AB for repair.
<i>No power to the system.</i>	Cable/cable installation failure.	Check the installation of cable/voltage drop. Alternatively send to Sacci Ryggsäckar AB for repair.
<i>Green display lights, but temperature of infusion solution bags exceeds 40 °C.</i>	Software error or short circuit on the circuit board.	Take the product out of service and send to Sacci Ryggsäckar AB for repair.

Product description

FUNCTIONAL DESCRIPTION

The product is a heat-regulated bag that holds three pieces of 1 L infusion solution bags and is intended for heating infusion solution to body temperature. The product is intended for installation in ambulance vehicles to provide body temperature infusion solution.

The product is an insulated bag with room for three 1 L infusion solutions. The product has a metal insert with a built-in temperature-controlling microcontroller which ensures that the temperature stays at $37\text{ }^{\circ}\text{C} \pm 2^{\circ}\text{C}$, where correct temperature specification gives green colour to the built-in display while red colour means that the temperature is outside the specification. The product also has a safety function in the form of mechanical overheating protection which shuts off the product if the temperature of the metal insert of the bag exceeds $60\text{ }^{\circ}\text{C}$, see figure 1.

During the heating phase, the metal insert will be warmer than $37\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$ to accelerate the heating of the infusion solution bags. When the infusion solution bags reach the desired temperature after about 2 hours, the product and the infusion solution bag will maintain the temperature of $37\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$.

Figure 1. Placement of display, microcontroller and overheating protection.

TECHNICAL DATA

Width 320 mm

Height 230 mm

Depth 100 mm

Weight of empty bag 980 grams

Weight of full bag (3 × 1 L infusion solution bags) 3780 grams

Material Acrylic coated polyamide fabric

Voltage 12 V

Power (20 °C) 75 W

Electric current 6,25 A

Electrical resistance 1,94 Ω ± 10%

Accessories and spare parts

CONTENT INSTALLATION KIT (ARTICLE NUMBER 009122)

Installation kit is supplied as an accessory and includes:

- » Contact for wall mounting
- » Switches for wall mounting
- » Fuse holder with fuses
- » Cables

MOUNTING INSTRUCTION FOR INSTALLATION KIT

See figure 2 for mounting instruction.

Preparation

1. Mount the customised aluminium rail (article number 112012) with a chassis knob in a suitable/desired location in the ambulance. No more than 1.4 metres from the power source which is limited by the cable length of 1.5 metres.
2. At a distance of max 60 cm from the bag, drill two round holes with a diameter of 26 mm and 23 mm respectively in the wall.
3. In the holes, the SELV contact and the switch included in the Installation kit are mounted.

Electrical connection

1. From the switch, which is mounted in the smaller of the pre-drilled holes, a separate supply cable with a cross-sectional area of 2.5 mm² (attached cable) – with the product as sole user – is drawn to the ambulance fuse holder or directly to the battery in the engine compartment.
2. If the ambulance has a special battery for healthcare equipment, the supply cable is drawn to this battery. The connection must be made with good quality 6.3 mm insulated cable shoe. For cars with 24 V DC, high quality converters must be installed. The cable length in the bag is 1.5 metres.
3. Connection must not be made to the car's cigarette lighter socket, as these usually cause large voltage drops after a period of use.

OPERATIONAL CONTROL


When the unit is connected to the socket, the display should be lit in red and the unit should start to heat up the infusion solution. When the desired temperature of 37 °C ± 2 °C has been reached, the display should light green.

If an external heat source is supplied and the temperature of the liquid becomes warmer than specified, the display will light in red. When the temperature drops, it will light green again.

Check the unit so that the heater becomes warm when connected. Apply a thermometer and read off so that the display shows green at the right interval. If the unit is out of order and the heater becomes too hot, an overheating protection will break the current.


In case of non-approved operational control, the product must be returned to Sacci Ryggsäckar AB for repair.

Figure 2. Mounting instructions for installation kit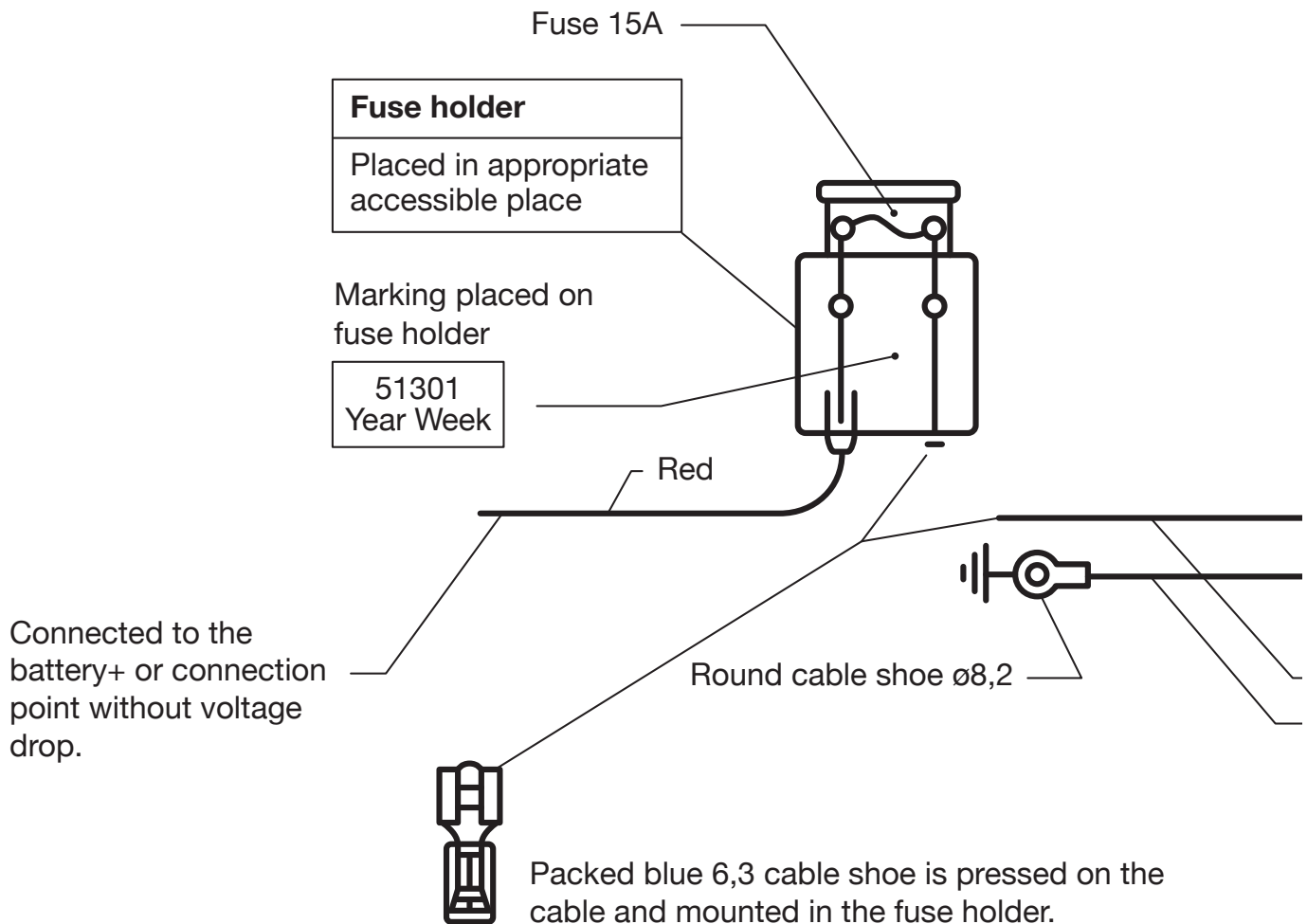


Switch with black casing and green lit spot (type 9).

- 1 + out
- 2 + in
- 3 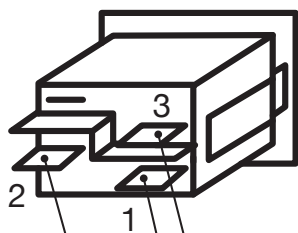

Mounting frame, hole $\varnothing 22$
 Power switch (possible hole 19,4x13)
 Mounting: First press the frame in the hole,
 followed by the switch

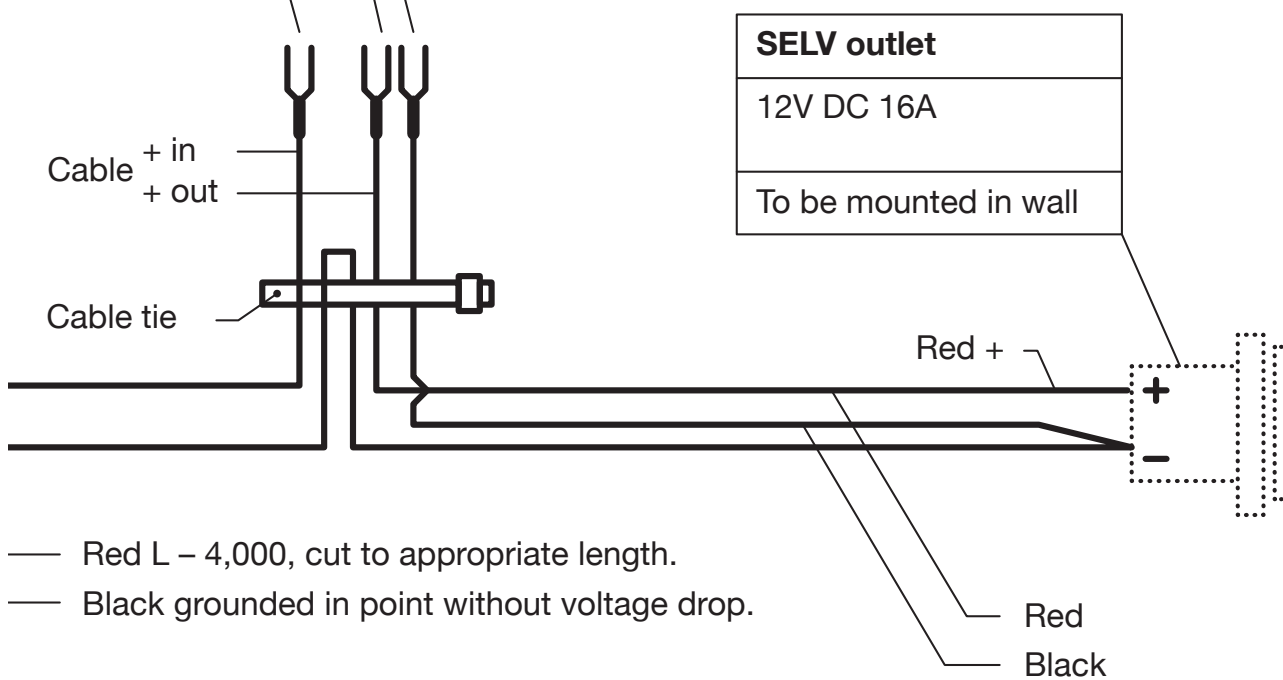- Red L – 4,000, cut to appropriate length.
- Black grounded in point without voltage drop.

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Sacci Ryggsäckar AB, Gamla Tunavägen 8