S-CUT RTL-501

User Manual

Warning!

The cutting edges of the circular blade is very sharp and must be handled with extreme care. Never use the tool if you are uncertain of how it is to be used. Store and use out of reach for children.

Intended use

The S-CUT cutting tool is to be used in rescue operations. The tool is used to quickly unfasten clothes, cut off straps, ropes and the like without having to change the person's position and thus risk further injury or unnecessary suffering.

Cleaning

Disassembly the complete tool. Do not touch the edge of the blade, it is very sharp! Clean all parts mechanically (i.e by using a brush or similar accepted method) with a soap solution and water. Disinfect all parts with an 70% alcohol solution and let dry. Make it to a practice to replace the plastic support regularly or when contaminated.

Maintenance

Make sure on a regularly basis that the tool is kept well cleaned and not damaged.

See illustration A for replacement of worn parts.

See illustration B for removal of tool parts.

See illustration C for assembly of tool parts.

Contact

Contact your dealer if you have further questions about this product.

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The cutting tool S-CUT is developed and manufactured in Sweden by ES Equipment AB. It is promoted and sold worldwide to all kind of rescue organisations. For further information, other models and accessories, see us at esequipment.se (Swedish) and s-cut.us.com (English). US Patent 7624507. European registered design.

Swedish webshop at s-cut.se. Contact us at kontaktaoss@esequipment.se

Tradenvägen 14
S-455 91 Munkedal • Sweden